Ovulationstest

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Instructions for use
EN-12 - EN-21

Mode d'emploi
FR-22 - FR-31

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Mode of operation	12
Safety instructions	13
Important notes on use	14
Test components	15
Determining the start of the test	15
Performing the test	17
Reading the test result	18
Use with the cyclotest® cycle computers	19
Frequently asked questions	20
Explanation of symbols	20
cyclotest® service hotline	21

Mode of operation

In-vitro diagnostic aid for self-testing to verify the presence of the ovulation hormone hLH in urine.

If you have decided to try for a baby, you want to know when your chances of conceiving are best. By measuring your body temperature regularly you will be able to pinpoint reliably your fertile phase.

The highly fertile phase and therefore the best time for conception is indicated by the ovulation hormone hLH, the presence of which can be detected with the help of this test.

Ovulation is the moment when the ovaries release a mature egg cell [ovum]. For women of sexually mature age, the levels of the hormone hLH (human luteinising hormone) in urine prior to the menopause are between 3 and 15 mIU/ml (Milli-International Units per millilitre).

About 24 hours before ovulation this rises to levels as high as 200 mIU/ml.

This test detects such a rise, indicating that fertilisation can soon take place. The cyclotest® Ovulation Test already indicates when the levels rise above 30 mIU/ml.

Since sperm cells can normally survive for several days inside a woman's body and take some time to migrate towards the ovum, this rise in hormone levels also marks the start of the phase in which sexual intercourse is most likely to lead to fertilisation of the ovum.

Then, between one and two days after ovulation, body temperature rises, marking the end of the fertile phase of the female cycle.

Safety instructions

This is an immunological test for detecting the presence of human luteinising hormone [hLH] in urine. Increased hLH levels in your urine mean that ovulation will probably (but not definitely) occur. Conversely, the absence of any rise of hLH does not always mean that ovulation will not occur.

Keep the test out of the reach of children.

Use the test immediately after opening the foil pouch. Do not use the test if the foil pouch is damaged!

This test is designed to be used once only. For this reason, read through the instructions for use carefully prior to use. The test stick will be rendered unusable if the test is incorrectly used.

Do not open the test stick.

For external use only.

The test materials used (such as antibodies) are potentially infectious material, but present no risk to health provided that all test components are used in accordance with the instructions for use.

Important notes on use

Use this test only when you are sure from your temperature measurement that you are already in your fertile phase.

You can use this test at any time of the day. The first (early-morning) urine sample is preferable however, as the hormone hLH is at its highest concentration in early-morning urine. If you carry out the test during the day, refrain from going to the toilet for 2-3 hours prior to the test and do not drink excessive amounts of fluid.

This fertility test determines the moment of ovulation and thus the fertile phase. However, because conception is influenced by many factors, you cannot assume from this that you will definitely become pregnant. If, in spite of using the test for several months, you fail to become pregnant, please consult your doctor.

For most women with a regular cycle 5 test sticks will be sufficient. If you have an irregular cycle, you may require a further pouch. Now and then a menstruation cycle may occur without a rise in hLH levels and consequently without ovulation. If, in spite of using the test correctly, you fail to detect any rise in hLH levels after 3 cycles, please consult your doctor. After you have stopped taking the pill, you

can determine with this test whether your cycle is once again accompanied by ovulation.

Medication containing hLH or hCG (human chorionic gonadotropin) may have a distorting effect on the test result. Likewise, the test will show incorrect results when used on the urine of pregnant women or menopausal women.

This test is not a suitable method of contraception!

Test components

Box, foil pouch with test stick and desiccant, instructions for use.

You will also require: a watch with a second hand and possibly a clean container.

Determining the start of the test

First you must determine the cycle length. To do so, count the days from the first day of menstruation up to the day before the next menstruation. The number of days counted is your cycle length.

Determining the start of the test

Cycle length (days)	Start of test (day)	Cycle length (days)	Start of test (day)
≤ 21	5	31	15
22	6	32	16
23	7	33	17
24	8	34	18
25	9	35	19
26	10	36	20
27	11	37	21
28	12	38	22
29	13	39	23
30	14	40	24


Example of a cycle length of 28 days: The table of cycle lengths above indicates the start of the test at 12 days after the start of the cycle. If the cycle starts on the 3rd of a month, performance of the Ovulation Test would start on the 15th of that month.

Sun	Mon	Tue	Wed	Thu	Fri	Sat
	1	2	③	4	5	6
7	8	9	10	11	12	13
14	15	16	17	18	19	20
21	22	23	24	25	26	27
28	29	30	31			

Start of cycle

Start of performance of test

Performing the test

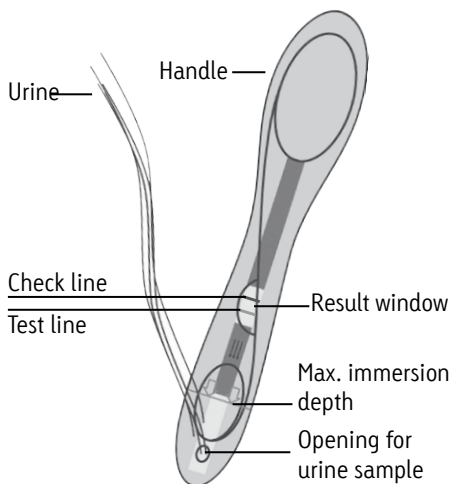Have ready a watch with a second hand and if necessary a clean container for your urine.

Open the foil pouch just before you are ready to perform the test.

Take the test stick out of the foil pouch. Please dispose of the enclosed desiccant unopened in the regular household waste.

Collect your urine sample in a clean, dry container. Hold the opening for the urine sample in the collected urine for approx. 3 seconds.

Important: Do not exceed the marking for the maximum immersion depth.

Alternatively, you can also hold the test stick directly in the urine stream for approx. 3 seconds (see illustration).

Now lay the test stick on the foil pouch for approx. 5 minutes.


 If 2 violet lines (upper check line and lower test line) appear in the result window and if the lower line is the same shade as or darker than the upper line, you are in your fertile phase.

 You may discontinue the test sequence here and keep the remaining test sticks for the next cycle.

 If 2 violet lines (upper check line and lower test line) appear in the result window and if the upper line is much darker than the lower line, this indicates that the test has been correctly carried out, but that you have not yet reached your fertile phase. The same applies if only one line appears.

In this case, you must carry out further tests on subsequent days until you obtain a positive result.


The test is classed as INVALID if no line appears in the result window after 5-10 minutes. Possible reasons for an invalid result may be a damaged foil pouch, incorrect storage or a mistake when performing the test. In this event, please keep all the test materials and contact the cyclotest® service hotline.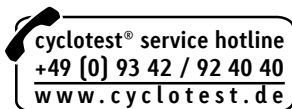

After a period of 10 minutes the test result can no longer be reliably read.

Dispose of the test sticks with your regular household waste.

If you are using one of the cyclotest® cycle computers (cyclotest® 2 plus or cyclotest® baby), you will recognise the start of your highly fertile phase by the baby symbols in the display. The cycle computer uses the stored data to calculate this phase and indicates this with two baby symbols appearing simultaneously in the display.

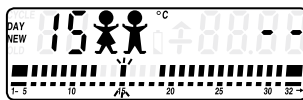Display: Highly fertile phase

In order to pinpoint even more accurately the time of optimum ability to conceive, you should now use the cyclotest® Ovulation Test. This test provides reliable indications as to whether and when ovulation occurs.

If the ovulation hormone has been detected, enter this information in the cycle computer. Refer to the notes in your cycle computer's instructions for use.

It is advisable to enter the result in the cycle computer on the very same day, as your highly fertile phase usually lasts only between one and two days.

For how long can I read the test result?

The test result must be read within 5 minutes. A positive result will not disappear. In the case of a negative test result, the stripe in the square test window may grow darker after the 5 minutes have been exceeded.

How long can I keep the cyclotest® Ovulation Test for?

If you store the cyclotest® test sticks in their sealed foil pouch at a temperature of between 4 °C and 30 °C, they can be used for an unlimited time up to the expiry date printed on the box.

I have recently come off the pill. How should I interpret my test results?

The body requires a certain length of time [usually 3, sometimes more cycles] to regenerate. As soon as your cycle has settled down again, you can use the cyclotest® Ovulation Test.

 This test complies with Council Directive 98/79/EC of 1998 on in-vitro diagnostics and bears the CE 0123 mark [TÜV SÜD Product Service GmbH].

cyclotest® service hotline

You have no doubt read our notes on using the cyclotest® Ovulation Test carefully and familiarised yourself with its features.

If you have any further questions, you can call our cyclotest® service hotline, which is open from Mondays to Thursdays from 7.30 am to 5.00 pm and on Fridays until 4.30 pm.


Your team of cyclotest® advisers is looking forward to taking your call.

Explanation of symbols

 Read the instructions for use carefully

 Use the test only once

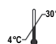 Store at between 4 and 30 °C

 Do not use the test after the expiry date

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© Copyright 2009 UEBE Medical GmbH
UEBE Medical GmbH