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cyclotest® baby

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DE-2 - DE-41

Instructions for Use
EN-42 - EN-81

Mode d'emploi
FR-82 - FR-121

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UEBE
Germany est. 1890

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Table of contents

Intended use	44
Safety instructions	45
Operating the unit	48
Unit description	48
Initial operation of the unit	50
Cycle start input forgotten	51
How to measure your wake-up temperature	51
How to read your display	54
Check displays	55
Ovulation symptoms	57
Input of ovulation symptoms	59
Data recall	61
Fertility profile	63
Evaluation of the data on a PC	63
Installing the USB drivers for cyclotest® baby	64
Installing the cyclotest® baby software	67
Read out the fertility profile	68
Battery display / battery replacement	70
Frequently asked questions	
When to measure and how often?	71
Turn off wake-up alarm	71
Pregnancy display	72
Fever and high temperature	73
Measurement button pressed without sensor at point of measurement	73
Irregular cycle	73
Time lag	73
The female cycle	74

Table of contents

Technical information	75
Failure and error messages	75
Customer service	75
Technical data	76
General provisions	78
Explanation of symbols	78
Disposal	78
Cleaning and maintaining the unit	79
Warranty	80
The cyclotest® service hotline	81

Thank you for choosing cyclotest® baby (also referred to in the following as the unit).

With cyclotest® baby you have acquired a modern unit for determining the most important phases in your cycle. Simple, fully automatic measurement of wake-up temperature enables you to pinpoint your fertile and infertile days. This eliminates completely the need to keep manual records.

Intended use

The unit is designed to measure the basal body temperature of women of child-bearing age and to determine the infertile, fertile and highly fertile phases in a woman's cycle. It is equipped with electronic circuitry for detecting, conditioning and evaluating basal body temperature, start of cycle and optionally LH hormone or cervical mucus input.

The maximum temperature determined using the sensor is automatically stored. The minicomputer in the unit calculates the fertile and infertile days from the acquired data. The respective phase within the cycle is displayed in the display.

Safety instructions

Optimum functioning of your cyclotest® baby cannot be guaranteed if you fail to observe the following safety instructions. Please read through the entire instructions for use carefully before using the unit. Follow the described procedure when using the unit on a daily basis.

- Check that the sensor is undamaged prior to measurement. Depending on where you use the sensor for measurement, a damaged sensor could cause unintentional injury.
- While cyclotest® baby still “knows” too little about you, i.e. has still not stored a sufficient amount of cycle values, a longer fertile phase will be displayed. The fertile phase prior to ovulation can be pinpointed more precisely only when at least six cycles have been stored.
- cyclotest® baby has a waking function: Once you have been discreetly woken by the unit, bring the unit into bed with you, place the sensor under your tongue, press the large Measurement button and wait until the unit beeps – done. All without having to get out of bed!
- Using the unit in the immediate vicinity of mobile phones, microwave appliances or other devices with strong electromagnetic fields may result in impaired functioning. Maintain a minimum distance of 3 m from such devices when using this unit.
- Do not under any circumstances open or modify the unit. Your warranty will be invalidated if you open the unit.

Safety instructions


- Do not leave the unit unsupervised in the hands of children. It contains small parts that could be swallowed. Danger of strangulation by the sensor cable.
- Please comply with the specified ambient conditions for measurement. See Technical data, Page 76.
- This temperature method does not provide protection against infectious diseases.
- These instructions are intended to help the user to use cyclotest® baby safely and efficiently and must be kept with the product and if necessary passed on.
- The unit must be used in accordance with the procedures contained in these instructions and must not be used for other purposes.
- The unit may only be connected to the devices described in these instructions for use.
- cyclotest® baby is a diagnostic unit for determining the fertile and infertile days in a woman's cycle and should not be used as a substitute for contraception methods [e.g. condom]. During the fertile phase you should, if you do not want to get pregnant, use the contraception method of your choice.
- The simultaneous use of cyclotest® baby and the pill is pointless. The pill is a contraceptive and renders the use of cyclotest® baby superfluous.

Safety instructions

- You are advised not to use the unit during irregular work and sleep periods [e.g. shift work] and in changeable cycles [e.g. puberty, illness, cycle lengths over 45 days].
- Please note that possible disturbance factors such as unaccustomed drinking of alcohol, exceptional stress and a change in environment can, in individual cases, influence the temperature. It may be advisable to omit measurement in such cases.
- It is essential to ensure correct functioning of cyclotest® baby that you perform the morning measurements of wake-up temperature before getting up. You should have slept for at least 5 hours and in the last hour prior to measurement you should not have got up.

Operating the unit

Unit description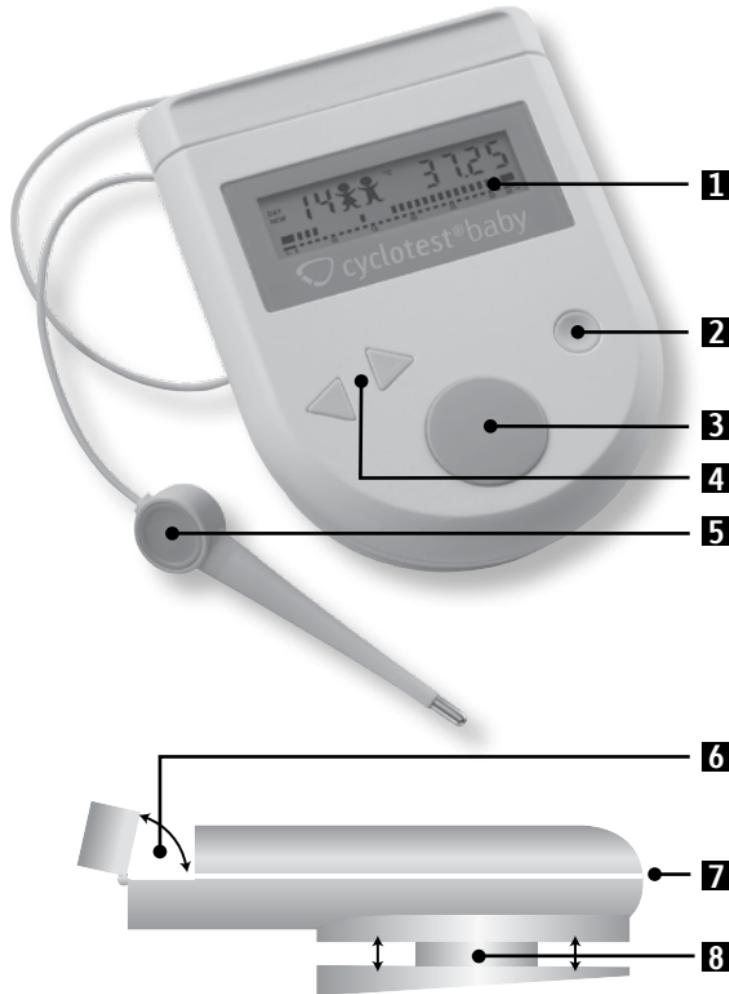

Operating the unit

1 Display

For displaying all the important data

2 Cycle Start button

Press once per cycle

3 Measurement button

For recording the wake-up temperature

4 Arrow buttons

For turning on the display and displaying

- the current cycle [right arrow]
- the previous cycles [left arrow]

5 Sensor

Highly sensitive with 1-metre cable – detects even minimal temperature fluctuations

6 Sensor compartment

Can be opened, for housing the sensor

7 USB port

For connecting to a PC

8 Cable reel

On the underside of the unit [pull to open]

Operating the unit

Initial operation of the unit

Press the small round Start button during initial operation to start the cycle and to start each subsequent new series of cycle measurements.

You enter the start of the cycle with the Start button. This should be done if possible on the first day of your menstruation. Press the Start button firmly and hold down until the input is confirmed with a beep.

You can enter the start of the cycle independently of a measurement, i.e. also in the afternoon or the evening.

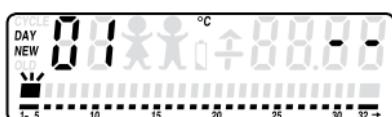
Fig. no. 1

Your display looks like this if the input has been successful.

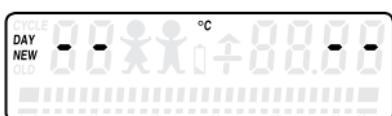
Fig. no. 2

If your display looks like this, the input has not been made.

In this case, wait until the unit has switched itself off and then repeat the input. Important: press the Cycle Start button firmly and for a sustained period.

Note: cyclotest® baby cannot store any measurement results if the start has not been input.

Operating the unit

Cycle start input forgotten

Press the small round Cycle Start button again firmly and for a sustained period. At the same time press the right Arrow button repeatedly until your current cycle day appears in the display (e.g. 02). Keep the Start button pressed until the input is confirmed with a beep. Your display should now look like this:

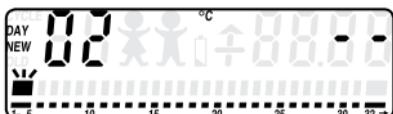
Fig. no. 3

Important: After a new cycle start you can no longer print out the last fertility profile. See Page 63.

How to measure your wake-up temperature

Once the cycle start has been successfully input, you can take your temperature measurements. Begin the morning after before getting up.

Note: At least 5 hours sleep needed prior to measurement. The measurement must be taken before you get up.

The sensor is located at the top in the sensor compartment. Pull the round cable reel on the back of the unit to extend the length of the cable sensor as required by uncoiling. This enables you to take your measurement comfortably while lying down. Then close the reel again.

You can take the measurement orally, rectally or vaginally, but you should then stick to the point of measurement you have chosen.

Operating the unit

Measuring under the arm (axillary) produces inaccurate results. It is therefore unsuitable for taking accurate measurements.

We recommend that you take the measurement under your tongue with your mouth closed. Place the sensor tip in one of the two heat pockets under your tongue to the left or right of the root of the tongue. The sensor must make good contact with the tissue. Close your mouth and breathe easily through your nose so that the measurement result is not compromised by inhaled air.

When the sensor tip is resting steadily and comfortably, press the large red Measurement button briefly.

Please be patient during the measurement. The sensor must warm up first [from room temperature to body temperature].

The unit confirms successful measurement [duration of measurement approx. 60-90 seconds] with a long beep and indicates the measured temperature in the display at top right. The unit switches off automatically after approx. 1 minute. Your measured value has been automatically stored.

The time of your first measurement remains stored in cyclotest® baby for all further measurements during a cycle. If, for example, you measured at 7 o'clock in the morning, you will be reminded of this the next morning with six short beeps. If you did not hear the first wake-up sound, you will be reminded after 10 and again after 30 minutes. We therefore recommend that you choose the time of your first measurement to match your daily rhythm.

cyclotest® baby has a time window of four hours and can store your measurements even if you measure up to two hours before or two hours after the time initially established. If you measure outside these four hours, the measurement is not stored and the following display appears:

Operating the unit

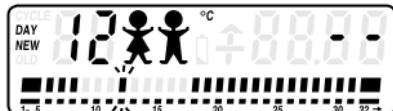
Fig. no. 4

If you break off the measurement too early, the unit will alert you with 3 short beeps that it was unable to store a measurement result because the warm-up phase was not completed. In this case, please measure again and remove the sensor only when you have heard a long beep. cyclotest® baby takes care of everything else in the background.

Proceed as in the first cycle when inputting a new cycle start and during the measurements.

Easy use in 3 steps!

1. On the first day of your period press the Cycle Start button until it beeps.
2. On the next morning of your cycle, directly after waking up and before getting up, measure your wake-up temperature. Put the sensor in your mouth and place it in one of the pockets under the tongue, press the Measurement button briefly and wait until it beeps.
3. Read off fertility status

Highly fertile phase

Fertile phase

Infertile phase

Note: The measurement period of +/- 2 hours is automatically established for the current cycle with the first measurement on the morning after the cycle start input.

Operating the unit

How to read your display

You can call up and view stored information in the display with the right Arrow button.

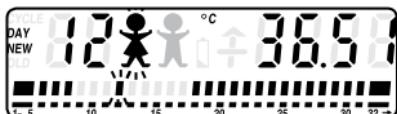
Fig. no. 5
Display example

This is how you read the display:

Current cycle

Your 12th day in the current cycle (also indicated by a flashing bar on the time axis)

You are in your fertile phase

On your 12th cycle day your measured wake-up temperature was 36.51 °C

Proceed as in the first cycle when inputting a new cycle start and during the measurements.

As from the second cycle start cyclotest® baby has “learned” a few things using your inputs and measurements in relation to the first cycle.

The expected end of the fertile phase in the current cycle is indicated – similarly to the previous cycle – in the display on the time axis.

Operating the unit

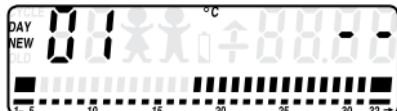
Fig. no. 6

A display looks like this after the start of the 2nd cycle

The expected infertile phase after ovulation begins in this example on the 18th cycle day: the bar chart for the time axis begins at 18.

cyclotest® baby can now display the highly fertile phase [2 permanent babies, Fig. no. 15]. In this phase, we recommend, if you want to have a baby, in addition to interpreting the temperature responses that you observe and input body symptoms.

Check displays

Note: The displays on the following pages are to be viewed as display examples.

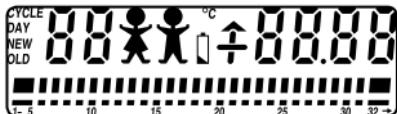
Fig. no. 7

The unit performs a function test after the Measurement button is pressed, all the display elements are shown.

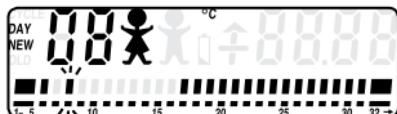
Fig. no. 8

DAY NEW: You are in your current cycle.

Operating the unit

Fig. no. 9

DAY OLD: When scrolling back, you have called up the measurement result of the 27th day of your previous cycle. Your current cycle measurement is not impaired by this.

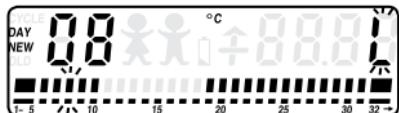
Fig. no. 10

L [Low]: Indicates that your wake-up temperature is below the measurement range of cyclotest® baby. A flashing "L" during the measurement indicates that the sensor is in its warm-up phase. See failure and error messages Page 75.

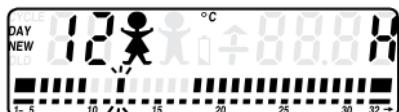
Fig. no. 11

H [High]: Indicates that your wake-up temperature is unusually high (e.g. fever). See failure and error messages Page 75.

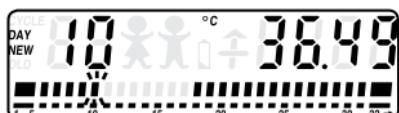
Fig. no. 12

10th cycle day, no "babies" mean infertile phase.

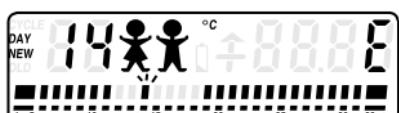
Fig. no. 13

E [Error]: Repeat the measurement if this letter is displayed. See failure and error messages Page 75.

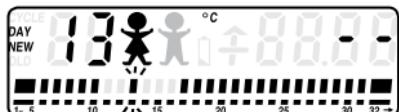
Fig. no. 14

"Baby": Alternately flashing "babies" mean "fertile phase".

Operating the unit

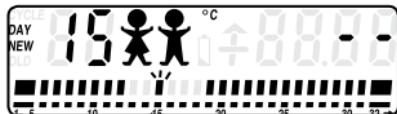
Fig. no. 15

"Babies": Two "babies" simultaneously mean "highly fertile phase". (display only from the second cycle).

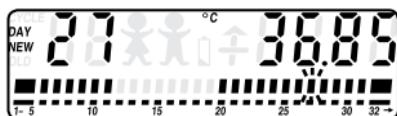
Fig. no. 16

"C": Indicates the measured wake-up temperature accurately in degrees Celsius to 2 decimal places.

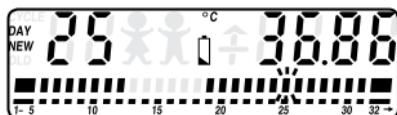
Fig. no. 17

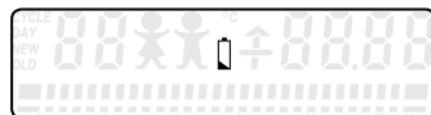
Fig. no. 18

"Battery": These displays appear during the measurement or when the data are called up if the battery power is low and the battery has to be replaced by UEBE Customer Service. See battery display / battery replacement Page 70.

Ovulation symptoms

What symptoms can be observed or verified in connection with imminent ovulation?

- Temperature rise
- Change in cervical mucus
- Increase in LH concentration in the urine

The temperature rise is monitored and evaluated by cyclotest® baby. The unit is able to store further symptoms which you input. This combined evaluation of temperature and symptom information is known as the symptothermal method.

Operating the unit

How important is the symptothermal method when you want to have a baby?

For couples who want to have a baby the time of the greatest capability of conception is important. The chance of pregnancy exists only when sperm cells capable of fertilising find a fertilisable egg cell.

The symptoms mentioned – nature of cervical mucus and increase in LH concentration – point to a fertilisable egg cell. Your fertilisability ends when the temperature high stabilises after ovulation.

In this event, no babies are indicated in the cyclotest® baby display.

Change in cervical mucus

The nature of the cervical mucus at the neck of the uterus changes shortly prior to ovulation. When this change is detected [spinnbarkeit – stretchability], the corresponding day must be marked in the current cycle.

A certain amount of experience is required to assess the cervical mucus. NFP advice centres [NFP = Natural Family Planning] can offer the necessary help in this respect. We will be pleased to send you a list of selected advice centres on request.

Operating the unit

Increase in LH concentration in the urine

An increased concentration of the ovulation hormone LH in the urine indicates that ovulation is imminent. This can be verified using cyclotest® Ovulation Test. If the test proves positive, enter the result on the corresponding day in the current cycle information.

Please use the test sticks only if you see 2 babies in the display [Fig. no. 15]. cyclotest® Ovulation Test is available from all good pharmacists [item no. 9035, PZN 01399381].

Input of ovulation symptoms

Note: The input stands either for cervical mucus or for LH concentration and must be made on the same day of the event.

1. Press and hold down the small round Start button. "01" appears.
2. Then press the Measurement button until "CY" appears in the display

Fig. no. 19
Your display now looks like this.

Operating the unit

3. Now release the Start button and keep the Measurement button pressed until the input is confirmed with a beep.

Please note! If you hold down the Start button for too long without pressing the Measurement button, a new cycle will be started.

It is only possible to input LH or cervical mucus information on the middle days of a cycle.

cyclotest® baby makes no distinction between whether you are observing the LH concentration or cervical mucus. You can only input one of the two. We recommend that you use cyclotest® Ovulation Test if you want to have a baby.

Pinpointing the highly fertile phase through the combined interpretation of temperature and hormone information has already helped many couples in fulfilling their wish to have a baby. The symptothermal method has had successful results even in cases that seem hopeless.

Operating the unit

Data recall

cyclotest® baby stores detailed data for the current cycle and the previous cycle and the key data from the last 12 cycles in the cycle overview.

Recall of detailed data in the current and previous cycles [unit is switched off]:

Briefly press the right Arrow button once. The current cycle day appears:

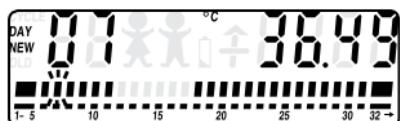
Fig. no. 20

Press the left Arrow button repeatedly. The previous days appear. You can scroll back in the current and previous cycles to the respective cycle start.

Display examples:

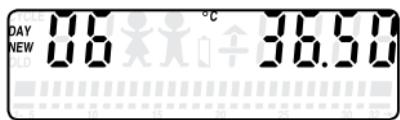
Fig. no. 21

DAY NEW: current cycle

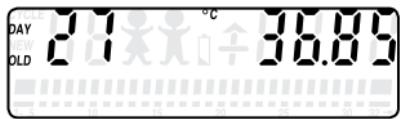
Fig. no. 22

DAY OLD: previous cycle

Operating the unit

Recall of cycle summaries for previous cycles [unit is switched off or on the current cycle day]:

Briefly press the right Arrow button twice. The following appears:

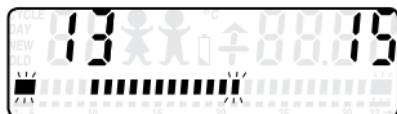
Fig. no. 23

First bar from the right shows the current cycle

This is how you read the overview:

- The digits on the left and right refer to the flashing cycle.
- Left digit: day of CY input if corresponding information was input
- Right digit: day of temperature high if a temperature high was detected

Press and hold down the large Measurement button: the temperature high is replaced by a display of the cycle length.

Fig. no. 24

Press the left or right Arrow button. The flashing bar in the bar chart moves and shows you which cycle you are in at the moment [right: current cycle, left = oldest stored cycle]. 12 cycles can be stored.

Operating the unit

Fertility profile

Where your gynaecologist in the past recommended that you kept a menstruation calendar, he/she today can analyse your cyclotest® baby fertility profile.


Your cyclotest® baby offers you this service:

- Display of all the information stored in the unit
- Evaluation of the data on a PC

Evaluation of the data on a PC

With the integrated USB port it is possible to read out the data of the current cycle from the unit and create your fertility profile as a PDF document.

Note: Once you have input a new cycle start, you can no longer display a temperature curve from the previous cycle. The data from the previous cycles are listed in table form below the current temperature curve.

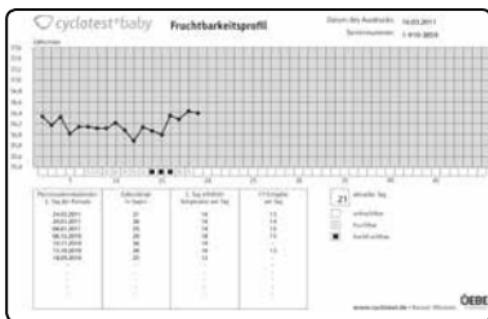Example of a fertility profile created with the cyclotest® baby software.

Operating the unit

You can print, save or mail these PDF documents.

To ensure that communication functions between cyclotest baby and the PC, you must install the drivers for the USB port and the cyclotest program. If you encounter problems during the installation procedure, helpful instructions are available on our website www.cyclotest.de in the “Downloads” section. You can also call the cyclotest® service hotline on +49 93 42 / 92 40 40.

Installing the USB drivers for cyclotest® baby

Supported operating systems: Windows Server 2008, Windows Server 2008 x64, Windows Vista, Windows Vista x64, Windows XP, Windows XP x64, Windows 2000, Windows Server 2003, Windows Server 2003 x64, Windows 98, Windows ME, Windows 7, Windows 7 x64, Windows 8

Before you can read out the data from the cyclotest® baby unit, you must first install the drivers for the USB port.

Important! Connect the supplied USB cable first to the PC and only then to the cyclotest® baby unit.

The “Wizard for finding new hardware” window will now appear automatically. If this window does not appear, try again by connecting the UCB cable to a different USB port.

Operating the unit

Please select the last selection option “Nein, diesmal nicht” and click on “Weiter”.

Insert the supplied cyclotest® CD in your PC's CD-ROM drive and click on “Weiter”. The driver is installed automatically from the inserted CD.

Complete the installation in the final window with “Fertig stellen”.

Then this window appears again. Please select the last selection option “Nein, diesmal nicht” and click on “Weiter”.

Operating the unit

In the next window click on “Weiter”. The driver is then installed automatically from the inserted CD.

Complete the installation in the final window with “Fertig stellen”.

This installation procedure may differ, depending on the operating system used.

You have successfully installed the driver for the USB port of your cyclotest® baby and can now install the cyclotest® baby software.

Help for problems encountered during the installation can be found on our homepage at www.cyclotest.de. Alternatively, call the cyclotest® service hotline on +49 9342 / 92 40 40.

Operating the unit

Installing the cyclotest® baby software

You will need a PDF Reader to display the data on your PC. A free version of Acrobat Reader is included on the cyclotest® CD. You can also download the PDF Reader from www.cyclotest.com.

Insert the cyclotest® baby software in your CD or DVD drive. The Auto-run function starts the Setup program automatically. If this does not happen, please start the Setup program manually via Windows Explorer. Double-click on [Drive]:\setup.exe – [Drive] here stands for the letter of your CD/DVD drive.

If the software is not already installed on your computer, the terms and conditions of use of the .NET Framework 2.0 used will appear first. Please confirm these with "Accept".

This brings up the message that .NET Framework 2.0 is being installed.

The installation may take a few minutes, depending on the hardware configuration.

Operating the unit

If .NET Framework has already been installed on your PC, the cyclotest® baby software will be installed straight away.

Confirm here with “Install”.

Your PC is now ready to read out the fertility profile.

Read out the fertility profile

1. You can only read out the fertility profile once the installation has been fully completed.
2. Important! Connect the supplied USB cable first to the PC and only then to the unit.

Attention: When pressing the large round test key do not wait longer than 1 second until pressing the cycle start key as otherwise you might start a new cycle! In case you started a new cycle accidentally, please call the cyclotest® service hotline.

3. Press the large round test key on your device briefly and then immediately press the small round cycle start key until USB appears on the display. Your cyclotest® monitor is now ready to transmit data to the PC!


Operating the unit

Note: When starting the programme, it will automatically search for updates. In case your software is not up to date, you will receive a warning.

If you wish to download and install the new version automatically, select "Ja".

- On your PC launch the software [in Programs, cyclotest] and click on the "Fruchtbarkeitsprofil auslesen" button. The program will now read out the data and display them on screen as a PDF file.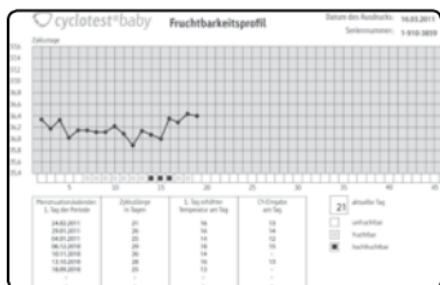

You can print, save or mail these PDF documents.

This eliminates completely the need to keep manual records.

Note: Read out your fertility profile before the next cycle starts. Once a new cycle start has been input, it is no longer possible to create a fertility profile from the previous cycle.

Operating the unit


Where your gynaecologist in the past recommended that you kept a menstruation calendar, he/she today can analyse your cyclotest® baby fertility profile.

By installing, copying or otherwise using this software you acknowledge the limitations of liability set out below.

UEBE Medical GmbH and its suppliers refuse - to the greatest possible extent permitted by the applicable law - all liability for any specific, accidental, indirect or consequential damage (including but not limited to damage from loss of prospective profits, interruption of business, loss of business information or any other economic loss) which occurs as a result of the use of the software or as a result of the fact that it cannot be used; this also applies if UEBE Medical GmbH was advised of the possibility of such damage beforehand. In this case the liability of UEBE Medical GmbH and its suppliers shall be limited in accordance with this agreement to the amount you actually paid for the software if the effect of this clause has not been specifically excepted by the applicable law.

Any lending or unauthorised copying, citing or transmission of the information contained on the data medium is a violation of the applicable law. This software license agreement is subject to German law.

Battery display / battery replacement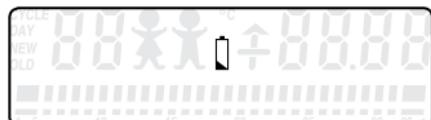

The battery symbol does not have a charge display.

If for measurement the battery symbol appears permanently in the display, this means that the battery needs to be replaced. Not to be confused with the display test when switching on, when all the symbols are displayed briefly. Continue measuring up to the infertile phase after ovulation. Then send the unit to UEBE Customer Service to have the battery replaced. Do not attempt to replace the battery yourself - risk of data being lost. See Customer service Page 65.

Frequently asked questions

When to measure and how often?

You can start your daily measurement the morning after the cycle start. What is important is that you have slept for at least 5 hours and you have not got up in the hour prior to measurement. No measurements are necessary during the infertile phase after ovulation. (No baby symbols are shown)

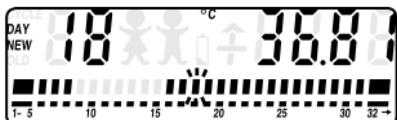
Fig. no. 25

You can now stop measuring until your next period.

If one or two measurements have been missed, this is not important. Simply continue measuring as normal on the next day. cyclotest® baby continues counting the days automatically. If there are too few measurements that can be evaluated in a cycle, the unit, to be on the safe side, displays a longer fertile phase than usual.

Important: With each initial measurement in the cycle you determine the time window of +/- 2 hours for the daily measurement. You can therefore decide anew for each cycle whether you want to stick with the selected time or opt for a new time.

Turn off wake-up alarm

If you do not want to be disturbed, e.g. you want to have a lie-in at the weekend, you can turn off the wake-up alarm. To do so, press the large round Measurement button once briefly when the unit is switched off. You will see the following display:

Frequently asked questions

Fig. no. 26
Wake-up alarm on (more bars)

Again press the large round Measurement button once briefly.

Fig. no. 27
Wake-up alarm off (fewer bars)

To turn the wake-up alarm back on, proceed as described above. The unit confirms the input with a beep.

Pregnancy display

cyclotest® baby is waiting for your cycle start. In this case, there are two possibilities:

- 1] You have forgotten to start a new cycle on the first day of menstruation.
- 2] If there is no menstruation, you may be pregnant.

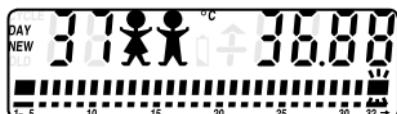
Fig. no. 28

Important: If within a cycle you observe inexplicable temperature responses which deviate from your previous cycles, call the cyclotest® service hotline.

Frequently asked questions

Fever and high temperature

An atypical increase in temperature is blanked out during the cycle recording. If there are not enough temperature values that can be evaluated during a cycle, the fertile phase is extended. See Fig. no. 11 "H".

Measurement button pressed without sensor at point of measurement

cyclotest® baby breaks off the measurement and signals this acoustically with three beeps and optically with an "E". See Fig. no. 13.

Irregular cycle

The unit displays a cycle length of 32 days in the bar chart. Counting is continued automatically for longer cycles.

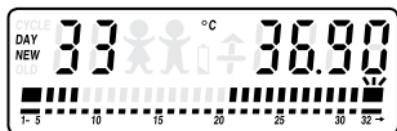
Fig. no. 29

You can recognise this by the flashing square in the bar chart, by the small arrow underneath and by the digit at top left.

In the event of irregular cycle lengths, a longer fertile phase is displayed.

Time lag

When travelling in Europe: If you are at the start of your cycle, communicate the start of your cycle to the cyclotest® baby with the small round button on the 1st day of your period and take a break from measuring until you have reached your destination and have acclimatised. Then resume measuring as normal. To ensure that the prognoses remain accurate, however, you should start the measurements no later than on the 5th cycle day.

Frequently asked questions

When travelling outside Europe:


The time differences on long-haul journeys are for the most part so great that your wake-up time is outside the “time window” in which cyclotest® baby accepts your measurement. In such cases please contact the cyclotest® service hotline in good time.

The female cycle

The female cycle is 28–29 days. Deviations of up to 5 days are considered to be within the normal standard range.

There are three distinct phases here:

- a relatively short infertile phase which begins with menstruation
- a fertile phase which ends shortly after ovulation
- a relatively long infertile phase after ovulation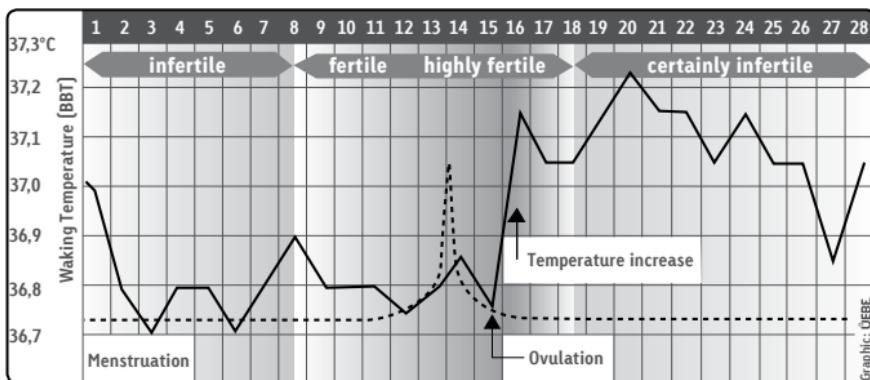

The temperature method, which is based on the comparative measurement of the wake-up temperature during a woman's individual cycle phases, allows conclusions to be drawn about the processes in a woman's body. The more temperature values and information on the cycle duration are available, the narrower the start and end of the fertile phase can be pinpointed.

Technical information

Failure and error messages

Failure encountered	Possible cause	Corrective action
Display "L"	- Sensor in the warm-up phase - Temperature too low - Possible technical fault	- Continue measuring until end of measurement - Repeat measurement - If "L" is permanently displayed, contact cyclotest service hotline
Display "H"	- Temperature too high - Fever - Illness - Possible technical fault	- If "H" is permanently displayed without fever, contact cyclotest service hotline
Display "E"	- Measured value not stable - Sensor removed before end of measurement	- Repeat measurement - If "E" is repeatedly displayed, contact cyclotest service hotline.

Customer service

The unit may only be repaired by the manufacturer or by an expressly authorised organisation.

Please contact:

UEBE Medical GmbH
Zum Ottersberg 9
97877 Wertheim, Germany
Phone: +49 [0] 9342/924040
Fax: +49 [0] 9342/924080
E-mail: info@uebe.com
Internet: www.uebe.com

Technical information

Technical data

Unit type:	Cycle temperature computer for recording basal body temperature
Method:	Syntothermal method modified on the basis of the recognised regulations of the WHO
Unit dimensions:	8 cm x 10.8 cm x 3.3 cm
Weight:	approx. 110 g
Housing material:	PVC-free, ABS-base plastic
Temperature sensor:	PVC-free, TPE-base flexible sensor
Measurement cable:	Special cable with strain-relief device (kevlar)
Sensor:	Thermistor
Display:	LCD display (liquid crystal display)
Display elements:	Day, temperature value in °C, baby symbols, battery symbol, time-axis display, fault indications, direction arrow
Temperature display:	Absolute in °C (non-extrapolating)
Measurement range:	35.5 - 38.0 °C
Measuring accuracy:	±0.1 °C at ambient temperature 10 °C to 35 °C, rel. air humidity up to 93 % non-condensing, air pressure 700 to 1040 hPa
Storage and transport conditions:	Ambient temperature -10 to +50 °C Rel. air humidity up to 93 % (non-condensing) Protect against moisture/humidity
Serial number:	The unit has a serial number which is identified with [SN]: please quote this number in the event of possible service enquiries.

Technical information

Battery:	RENATA type CR2450N button cell, battery for approx. 500 measurements, battery to be replaced only by UEBE Medical GmbH. The unit may only be repaired by the manufacturer or by an expressly authorised organisation. Do not open the unit, as this will invalidate the warranty.
Service:	Should your unit show evidence of a fault, we ask you not to attempt repairs yourself, but instead to send the unit with a short description of the fault directly to UEBE Medical GmbH for inspection, as unit parts cannot be repaired individually.
Calibration:	The unit does not have to be calibrated; the metrological accuracy of the unit will be automatically checked when the battery is replaced by UEBE Medical GmbH.
Protection class IP:	IP 20: Protected against solid foreign particles with a diameter of more than 12.5 mm, no protection against water.
Expected service life:	5 years

Additional equipment connected to medical electrical equipment must comply with the respective IEC or ISO standards [e.g. IEC 60950 for data-processing equipment]. Furthermore, all configurations shall comply with the normative requirements for medical systems (see IEC 60601-1-1 or Section 16 of the 3rd edition of IEC 60601-1, respectively). Anybody connecting additional equipment to medical electrical equipment is a system configurer and is therefore responsible for ensuring that the system complies with the normative requirements for systems. Attention is drawn to the fact that local laws take priority over the above-mentioned normative requirements. If in doubt, consult your local specialised dealer or the Technical Service department.

General provisions

Explanation of symbols


Degree of protection against electric shock: TYP BF

Please observe operating instructions!

CE 0123 This product complies with the Council Directive 93/42/EC from 5 September 2007 regarding medical devices and bears the mark CE 0123 [TÜV SÜD Product Service GmbH].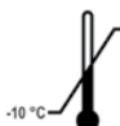

Storage and transportation conditions
Ambient temperature -10 to +50 °C

Protect against moisture/humidity
Relative air humidity up to 93 %

Keep dry

Manufacturer

Disposal

Batteries and technical appliances must not be disposed of with domestic waste, but should be handed in at the appropriate collection and disposal points.

Cleaning and maintaining the unit

The flexible part of the sensor is waterproof, the rest of the sensor and the unit may only be wiped with a moistened cloth.

The device is not waterproof and must not be immersed in water! Clean the unit with a moistened cloth and a mild cleaning agent. You may use ethyl alcohol [available from all good pharmacies] to disinfect the sensor.

Do not expose the thermometer to extreme heat [blazing sun, heating, hot water, etc.].

Warranty

The device has been manufactured and tested with great care. However, in the unlikely event of a defect being detected after delivery, we provide warranty for new devices in accordance with the following terms and conditions:

1. During the warranty period of 2 years from the date of purchase we reserve the right either to repair any such defect at our expense or to supply a perfect replacement unit. The cost of returning the unit to our factory shall be borne by the sender. UEBE shall refuse to accept return deliveries that have not been paid for by the sender.
2. Excluded from the warranty are parts subject to normal wear and tear as well as damage caused by non-compliance with the instructions for use, improper handling [e.g. unsuitable power sources, breakages, leaking batteries] and/or disassembly of the unit by the purchaser. and/or disassembly of the unit by the purchaser. Furthermore, no claims for damages against us are substantiated by the warranty.
3. Warranty claims can only be advanced in the warranty period and by presenting proof of purchase. In the event of a warranty claim, the unit must be sent to the following address together with the proof of purchase and a description of the complaint:

UEBE Medical GmbH
Service-Center
Zum Ottersberg 9
97877 Wertheim
Germany

The cost of returning the unit to our factory shall be borne by the sender. UEBE shall refuse to accept return deliveries that have not been paid for by the sender.

The cyclotest® service hotline

4. In the event of defectiveness of the goods, the statutory rights of the purchaser to claim against the seller in accordance with § 437 German Civil Code are not limited by the warranty.


Please note: In the event of a warranty claim it is essential to attach the proof of purchase.

Dear Reader,

Ensure you read our notes on using the cyclotest® baby carefully and familiarised yourself with its features.

Our description corresponds to the normal cycle response. While dealing with your cyclotest® baby you may notice initially some readings and information in the dialogue display that appear unfamiliar to you, or you may have questions about handling the unit.

In such cases you can call our cyclotest® service hotline, which is open from Mondays to Thursdays from 8.00 am to 5.00 pm and on Fridays until 4.30 pm.

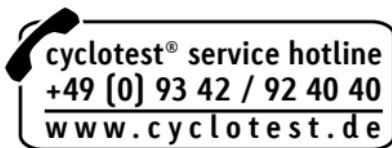Your team of cyclotest® advisers is looking forward to taking your call.

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© Copyright 2014 UEBE Medical GmbH