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cyclotest® 2 plus

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DE-2 - DE-43

Instructions for Use
EN-44 - EN-85

Mode d'emploi
FR-86 - FR-127

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Table of contents

Intended use	46
Safety instructions	47
Important advice for users	48
Important technical details	50
Operating the unit	52
Unit description	52
Easy use in 3 steps!	54
Input the start of your cycle	55
Cycle start input forgotten	56
How to measure your wake-up temperature	57
Alarm function	61
How to read your display	62
Check displays	64
Ovulation symptoms	66
Input of ovulation symptoms	68
Data recall	69
Fertility profile	71
Evaluation of the data on a PC	71
Battery display / battery replacement	72
Frequently asked questions	73
Fever / increased temperature and irregularities	73
Irregular cycle	73
Time lag	74
Turn off wake-up alarm	74
Measurement button pressed without sensor at point of measurement	75
Start of cycle input incorrectly	75
Pregnancy display	75

Table of contents

cyclotest® 2 plus only indicates the fertile phase	76
When to measure and how often?	76
The female cycle	77
Technical information	78
Failure and error messages	78
Customer service	78
Technical data	79
General provisions	81
Explanation of symbols	81
Disposal	81
Cleaning and maintaining the unit	82
Warranty	83
The cyclotest® service hotline	84

Thank you for choosing cyclotest® 2 plus [also referred to in the following as the unit].

With cyclotest® 2 plus you have acquired a modern unit for determining the most important phases in your cycle. Simple, fully automatic measurement of wake-up temperature enables you to pinpoint your fertile and infertile days. This eliminates completely the need to keep manual records.

The unit is designed to measure the basal body temperature of women of child-bearing age and to determine the infertile, fertile and highly fertile phases in a woman's cycle. It is equipped with electronic circuitry for detecting, conditioning and evaluating basal body temperature, start of cycle and optionally LH hormone or cervical mucus input.

The maximum temperature determined using the sensor is automatically stored. The minicomputer in the unit calculates the fertile and infertile days from the acquired data. The respective phase within the cycle is indicated in the display.

Optimum functioning of your cyclotest® 2 plus cannot be guaranteed if you fail to observe the following safety instructions. Please read through the entire instructions for use carefully before using the unit. Follow the described procedure when using the unit on a daily basis.

- **You must use additional protection during your fertile phase**

cyclotest® 2 plus is a diagnostic unit for determining the fertile and infertile days in a woman's cycle and should not be used as a substitute for contraception methods. During the fertile phase you should, if you do not want to get pregnant, use the contraception method of your choice [e.g. cyclotest® "Natural Love" condoms, central pharmaceutical number 10186690].

- **Control your cycle without hormones**

It is pointless to use cyclotest® 2 plus and hormonal contraceptives at the same time. Hormonal contraceptives influence your natural monthly cycle and thus the function of cyclotest® 2 plus, too.

- **Caution in the event of irregularities**

You are advised not to use the unit during irregular work and sleep periods [e.g. shift work] and in changeable cycles [e.g. puberty, illness, cycles lasting more than 45 days, menopause]. See page 73. Ideal for use in women after puberty up until the onset of the menopause with regular working and sleeping hours.

Important advice for users

- Check that the sensor **is undamaged prior to measurement**. Depending on where you use the sensor for measurement, a damaged sensor could cause unintentional injury.
- It is essential to ensure correct functioning of cyclotest® 2 plus that you perform the morning measurements of wake-up temperature before getting up. You should have **slept for at least 5 hours** and in the last hour prior to measurement you should not have got up.
- While cyclotest® 2 plus still “knows” too little about you, i.e. has still not stored a sufficient amount of cycle values, **a longer fertile phase** will be displayed for your assurance. The fertile phase **prior to ovulation** can be pinpointed more precisely only when at least six cycles have been stored.
- Please note that possible **disturbance factors** such as unaccustomed drinking of alcohol, exceptional stress and a change in environment can, in individual cases, **influence the temperature**. It may be advisable to omit measurement in such cases. See Page 73.
- If you discover that you have an **unusual increase in temperature** due to illness, we advise not taking any further measurements until you feel better.
- **After breastfeeding**, you should only start taking measurements once you are menstruating regularly again.


- This contraception method **does not provide protection against infectious diseases**.
- The device **does not actively intervene in your cycle**; it only evaluates temperature parameters and indicates your current status in the cycle.
- Do not leave the unit unsupervised in the hands of children. It contains small parts that could be swallowed. **Danger of strangulation** by the sensor cable.

Important technical details

- Using the unit in the immediate vicinity of mobile phones, microwave appliances or other devices with strong electromagnetic fields may result in impaired functioning. Maintain a minimum distance of 3 m from such devices when using this unit.
- Never open the device or tamper with it. This is a medical device and may only be opened by authorised specialists. The guarantee will become void if the device is opened.
- The device may only be operated with genuine spare parts and accessories. The guarantee shall expire in the event of damage caused by third-party accessories!
- The device contains sensitive parts and must be protected from severe fluctuations in temperature, as well as from humidity, dust and direct sunlight.
- Do not expose the device to extreme heat (bright sunlight, heat, hot water, etc.).
- The device is not shock-proof or impact-resistant. After serious falls or impacts, we advise that the device be checked over to ensure that the display indicator is intact and accurate.
- Comply with the specified operating conditions for measurement. See Technical data, Page 79.

- These instructions are intended to help the user to use cyclotest® 2 plus safely and efficiently and must be kept with the product and if necessary passed on.
- The unit must be used in accordance with the procedures contained in these instructions and must not be used for other purposes.
- All serious incidents which occur in connection with the device must be reported to the manufacturer and to the responsible authorities in the member state where the user is domiciled.

Unit description

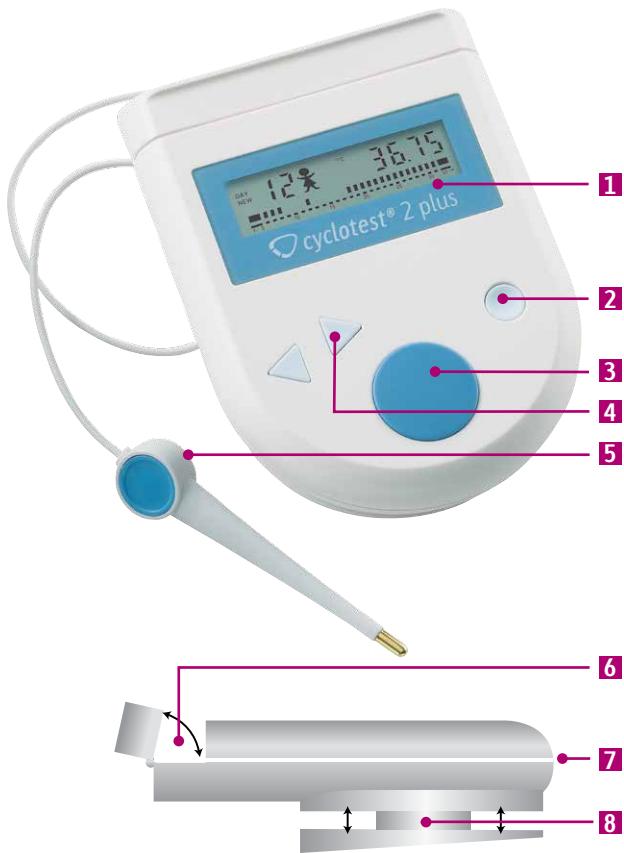1 Display

For displaying all the important data

2 Cycle Start button

Press once per cycle

3 Measurement button

For recording the wake-up temperature

4 Arrow buttons

For turning on the display and displaying
- the current cycle (right arrow)
- the previous cycles (left arrow)

5 Sensor

Highly sensitive with 1-metre cable – detects even minimal temperature fluctuations

6 Sensor compartment

Can be opened, for housing the sensor

7 USB port

For connecting to a PC [additional software required]

8 Cable reel

On the underside of the unit [pull to open]

Easy use in 3 steps!

1 Inputting the start of your cycle

On the first day of your period, press and hold the Cycle Start button until you hear a beep confirming the input.

2 Measuring your waking temperature

On the next morning of your cycle, directly after waking up and before getting up, measure your wake-up temperature. Put the sensor in your mouth and place it in one of the pockets under the tongue, press the Measurement button briefly and wait until it beeps.

3 Read off fertility status

-  Highly fertile phase
-  Fertile phase
-  Infertile phase

Note: The measurement period of +/- 2 hours is automatically established for the current cycle with the first measurement on the morning after the cycle start input.

1 Input the start of your cycle

Press the small round Start button during initial operation to start the cycle and to start each subsequent new series of cycle measurements.

You enter the start of the cycle with the Start button. This should be done if possible on the first day of your menstruation. Press the Start button firmly and hold down until the input is confirmed with a beep.

You can enter the start of the cycle independently of a measurement, i.e. also in the afternoon or the evening.

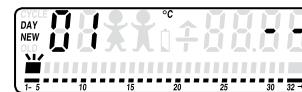
Fig. no. 1
Your display looks like this if the input has been successful.

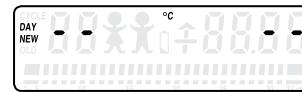
Fig. no. 2
If your display looks like this, the input has not been made.

In this case, wait until the unit has switched itself off and then repeat the input. Important: press the Cycle Start button firmly and for a sustained period.

Note: cyclotest® 2 plus cannot store any measurement results if the start has not been input.

Cycle start input forgotten

Press the small round Cycle Start button again firmly and for a sustained period. At the same time press the right Arrow button repeatedly until your current cycle day appears in the display [e.g. 02]. Keep the Start button pressed until the input is confirmed with a beep. Your display should now look like this:

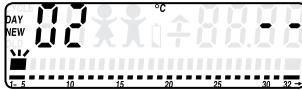
Fig. no. 3

Important: After a new cycle start you can no longer print out the last fertility profile. See Page 71.

2 How to measure your wake-up temperature

Once the cycle start has been successfully input, you can take your temperature measurements. Begin the morning after before getting up. Select the time of your first measurement to correspond to your daily routine.

1. Determining the measurement period:

cyclotest® 2 plus has a measurement window of four hours. With the first measurement on the morning after you inputted the start of your cycle, the measurement period of +/- 2 hours is automatically determined for the current cycle. If, for example, you took a measurement at 7.00 a.m., the next morning six short beeps will remind you to take a measurement at this time (also see page 61). For this cycle, you can take measurements from 5.00 a.m. to 9.00 a.m.

Note:

- Measurements taken as close as possible to the same time period optimise the evaluation.
- You have to get at least five hours of sleep before taking a measurement.
- You must take the measurement before getting up.

2. Unwinding the sensor cable:

The sensor is located at the top in the sensor compartment. Pull the round cable reel on the back of the unit to extend the length of the cable sensor as required by uncoiling. This enables you to take your measurement comfortably while lying down. Then close the reel again.

3. Selecting the measurement location:

You can take the measurement orally, rectally or vaginally, but you should then stick to the point of measurement you have chosen. Measuring under the arm [axillary] produces inaccurate results. It is therefore unsuitable for recording the basal temperature.

4. Measurement process:

We recommend that you take the measurement under your tongue with your mouth closed. Place the sensor tip in one of the two heat pockets under your tongue to the left or right of the root of the tongue. The sensor must make good contact with the tissue. Close your mouth and breathe easily through your nose so that the measurement result is not compromised by inhaled air.

5. Starting the measurement:

When the sensor tip is resting steadily and comfortably, press the large red Measurement button briefly.

Please be patient during the measurement. The sensor must warm up first [from room temperature to body temperature].

6. End of measurement:

The unit confirms **successful measurement** (duration of measurement approx. 60-90 seconds) **with a long beep** and indicates the measured temperature in the display at top right. The unit switches off automatically after approx. 1 minute.

If you **break off the measurement too early**, the unit will alert you **with 3 short beeps** that it was unable to store a measurement result because the warm-up phase was not completed. In this case, please measure again and remove the sensor only when you have heard a long beep. cyclotest® 2 plus takes care of everything else in the background.

Your measured value has been automatically stored.

Once you have taken the measurement, cyclotest® 2 plus indicates your fertility status for the current day.

Operating the unit

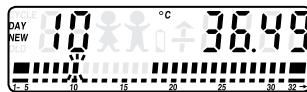
Fig. no. 12 no "babies" mean **infertile phase**.

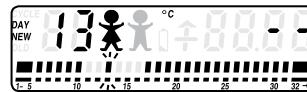
Fig. no. 14 Alternately flashing "babies" mean "**fertile phase**".

6. Measured value is saved:

If you measure outside these four hours, the measurement is not stored and the following display appears:

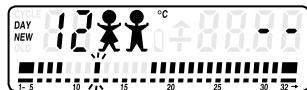
Fig. no. 4

Further information from the second cycle

Proceed as in the first cycle when inputting a new cycle start and during the measurements.

As from the second cycle start cyclotest® 2 plus has "learned" a few things using your inputs and measurements in relation to the first cycle.

The expected end of the fertile phase in the current cycle is indicated – similarly to the previous cycle – in the display on the time axis.

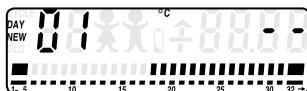
Fig. no. 6
A display looks like this after the start of the 2nd cycle

Operating the unit

The expected infertile phase after ovulation begins in this example on the 18th cycle day: the bar chart for the time axis begins at 18.

cyclotest® 2 plus can now display the highly fertile phase [2 permanent babies]. In this phase, we recommend in addition to interpreting the temperature responses that you observe and input body symptoms. (See Page 68)

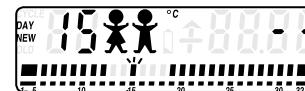
Fig. no. 15 "Babies": Two "babies" at the same time mean "**highly fertile phase**" (indication from the second cycle only).

Alarm function

Attention: Select the time of your first measurement to correspond to your daily routine.

cyclotest® 2 plus has a measurement window of four hours. With the first measurement on the morning after you inputted the start of your cycle, the measurement period of +/- 2 hours is automatically determined for the current cycle. The time of your first measurement remains stored in cyclotest® 2 plus for all further measurements during a cycle. If, for example, you measured at 7 o'clock in the morning, you will be reminded of this the next morning with six short beeps.

If you did not hear the first wake-up sound, you will be reminded after 10 and again after 30 minutes.

Note: During your infertile phase following ovulation, you no longer receive any measurement reminders. No further measurements are required until you start your next cycle.

3 How to read your display

You can call up and view stored information in the display with the right Arrow button.

This is how you read the display:

Current cycle Your 12th day in the current cycle [also indicated by a flashing bar on the time axis]

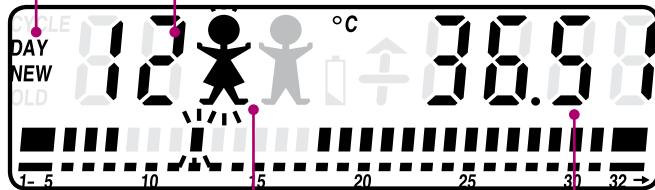
Fig. no. 5 Display example

You are in your fertile phase

On your 12th cycle day your measured wake-up temperature was 36.51 °C

Indicates which cycle you are currently in.

DAY NEW = current cycle

DAY OLD = previous cycles

Indicates which day you are currently on. Also indicated by means of the flashing bar on the time axis.


Indicates which fertility status you are currently in.
TWO BABIES at the same time = highly fertile phase
ONE BABY alternating = fertile phase
NO BABIES = infertile phase

Indicates your measured waking temperature in °C to two decimal places.

Grey bars [faded-out bars] indicate your expected fertile phase.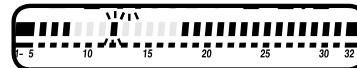

Black bars [faded-in bars] indicate your expected infertile phase.

Check displays

Note: The displays on the following pages are to be viewed as display examples.

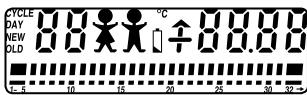
Fig. no. 7

The unit performs a function test after the Measurement button is pressed, all the display elements are shown.

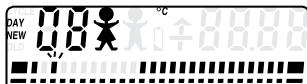
Fig. no. 8

DAY NEW: You are in your current cycle.

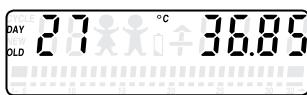
Fig. no. 9

DAY OLD: When scrolling back, you have called up the measurement result of the 27th day of your previous cycle. Your current cycle measurement is not impaired by this.

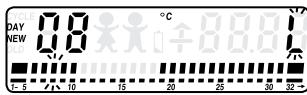
Fig. no. 10

L [Low]: Indicates that your wake-up temperature is below the measurement range of cyclotest® 2 plus. A flashing "L" during the measurement indicates that the sensor is in its warm-up phase. See failure and error messages Page 78.

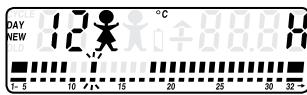
Fig. no. 11

H [High]: Indicates that your wake-up temperature is unusually high (e.g. fever). See failure and error messages Page 78.

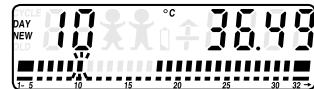
Fig. no. 12

10th cycle day, no "babies" mean infertile phase.

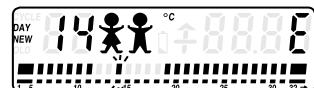
Fig. no. 13

E [Error]: Repeat the measurement if this letter is displayed. See failure and error messages Page 78.

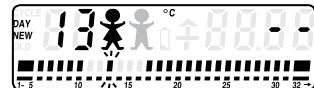
Fig. no. 14

"Baby": Alternately flashing "babies" mean "fertile phase".

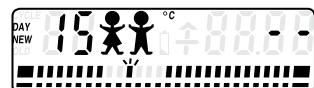
Fig. no. 15

"Babies": Two "babies" simultaneously mean "highly fertile phase". (display only from the second cycle).

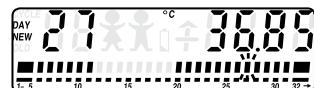
Fig. no. 16

°C: Indicates the measured wake-up temperature accurately in degrees Celsius to 2 decimal places.

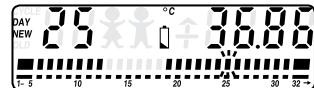
Fig. no. 17

"Battery": These displays appear during the measurement or when the data are called up if the battery power is low and the battery has to be replaced by UEBE Customer Service. See battery display / battery replacement Page 72.

Fig. no. 18

"Arrow": indicates that your temperature is still increasing during the measurement.

Ovulation symptoms

What symptoms can be observed or verified in connection with imminent ovulation?

- Temperature rise
- Change in cervical mucus
- Increase in LH concentration in the urine

Temperature rise

The temperature rise is monitored and evaluated by cyclotest® 2 plus. The unit is able to store further symptoms which you input. This combined evaluation of temperature and symptom information is known as the symptothermal method.

The chance of pregnancy exists only when sperm cells capable of fertilising find a fertilisable egg cell.

The symptoms mentioned – nature of cervical mucus and increase in LH concentration – point to a fertilisable egg cell. Your fertilisability ends when the temperature high stabilises after ovulation.

In this event, no babies are indicated in the cyclotest® 2 plus display.

Change in cervical mucus

The nature of the cervical mucus at the neck of the uterus changes shortly prior to ovulation. When this change is detected [stretchability], the corresponding day must be marked in the current cycle.

A certain amount of experience is required to assess the cervical mucus. NFP advice centres [NFP = Natural Family Planning] can offer the necessary help in this respect. We will be pleased to send you a list of selected advice centres on request.

Increase in LH concentration in the urine

An increased concentration of the ovulation hormone LH in the urine indicates that ovulation is imminent. This can be verified using cyclotest® Ovulation Test. If the test proves positive, enter the result on the corresponding day in the current cycle information.

Please use the test sticks only if you see 2 babies in the display [Fig. no. 15, page 63]. cyclotest® Ovulation Test is available from all good pharmacists [item no. 9035, PZN 01399381].

Input of ovulation symptoms

Note: The input stands either for cervical mucus or for LH concentration and must be made on the same day of the event.

1. Press and hold down the small round Start button. "01" appears.
2. Straight afterwards press the large round Measurement button until "CY" appears on the right in the display.

Fig. no. 19
Your display now looks like this.

3. Now release the Start button and keep the Measurement button pressed until the input is confirmed with a beep.

Please note! If you hold down the Start button for too long without pressing the Measurement button, a new cycle will be started.

It is only possible to input LH or cervical mucus information on the middle days of a cycle.

cyclotest® 2 plus makes no distinction between whether you are observing the LH concentration or cervical mucus. You can only input one of the two. We recommend that you use the cyclotest® Ovulation Test [Art. Nr. 9035, PZN 01399381] to identify the ovulation hormone.

Data recall

cyclotest® 2 plus stores detailed data for the current cycle and the previous cycle and the key data from the last 12 cycles in the cycle overview.

Recall of detailed data in the current and previous cycles (unit is switched off):

Briefly press the right Arrow button once. The current cycle day appears:

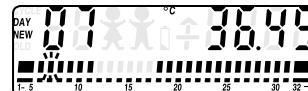
Fig. no. 20

Press the left Arrow button repeatedly. The previous days appear. You can scroll back in the current and previous cycles to the respective cycle start.

Display examples:

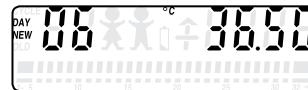
Fig. no. 21
DAY NEW: current cycle

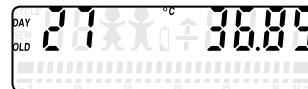
Fig. no. 22
DAY OLD: previous cycle

Recall of cycle summaries for previous cycles (unit is switched off or on the current cycle day):

1. Briefly press the right Arrow button twice. The following appears:

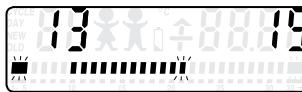
Fig. no. 23
First bar from the right shows the current cycle

This is how you read the overview:

- The digits on the left and right refer to the flashing cycle.
- Left digit: day of CY input if corresponding information was input
- Right digit: day of temperature high if a temperature high was detected

Press and hold down the large Measurement button: the temperature high is replaced by a display of the cycle length.

Fig. no. 24

Press the left or right Arrow button. The flashing bar in the bar chart moves and shows you which cycle you are in at the moment [right: current cycle, left = oldest stored cycle]. 12 cycles can be stored.

Fertility profile

Where your gynaecologist in the past recommended that you kept a menstruation calendar, he/she today can analyse your cyclotest® 2 plus fertility profile.

Your cyclotest® 2 plus offers you this service:


- Display of all the information stored in the unit
- Evaluation of the data on a PC

Evaluation of the data on a PC

To ensure communication between cyclotest® 2 plus and your PC, you will need the cyclotest® control software. This software contains the drivers for the USB port and the cyclotest® program for evaluating the stored data. The cyclotest® control software with USB connecting cable is available as an accessory from all good pharmacies. Art. no. 065501, PZN 05103265.

With the integrated USB port and additional software it is possible to read out the data of the current cycle from the unit and create your fertility profile as a PDF document.


Note: Once you have input a new cycle start, you can no longer display a temperature curve from the previous cycle. The data from the previous cycles are listed in table form below the current temperature curve.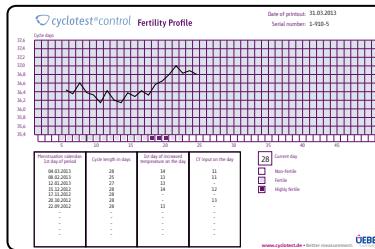

Example of a fertility profile created with the cyclotest® control software.

You can print, save or mail these PDF documents.

Battery display / battery replacement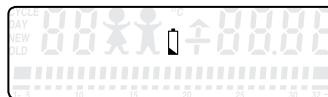

The battery symbol does not have a charge display

If for measurement the battery symbol appears permanently in the display, this means that the battery needs to be replaced. Not to be confused with the display test when switching on, when all the symbols are displayed briefly. Continue measuring up to the infertile phase after ovulation. Then send the unit to UEBE Customer Service to have the battery replaced. Do not attempt to replace the battery yourself - risk of data being lost. See Customer service Page 78.

Fever / increased temperature and irregularities

An atypical increase in temperature is blanked out during the cycle recording. If this is due to illness, we advise not taking any further measurements until you feel better. You should also refrain from taking any measurements in the event of irregularities [e.g. getting too little sleep, partying, drinking alcohol, feeling stressed, eating a large meal late at night, taking medication which alters your temperature, experiencing illness and fever]. If there are not enough temperature values that can be evaluated during a cycle, the fertile phase is extended. See Fig. no. 11 "H".

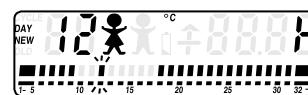
Fig. no. 11

H [High]: Indicates that your wake-up temperature is unusually high [e.g. fever]. See failure and error messages Page 78.

Irregular cycle

The unit displays a cycle length of 32 days in the bar chart. Counting is continued automatically for longer cycles.

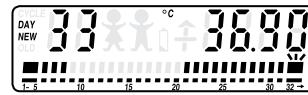
Fig. no. 29

You can recognise this by the flashing square in the bar chart, by the small arrow underneath and by the digit at top left.

In the event of irregular cycle lengths, a longer fertile phase is displayed.

Time lag

When travelling in Europe: If you are at the start of your cycle, communicate the start of your cycle to the cyclotest® 2 plus with the small round button on the 1st day of your period and take a break from measuring until you have reached your destination and have acclimatised. Then resume measuring as normal. To ensure that the prognoses remain accurate, however, you should start the measurements no later than on the 5th cycle day.

When travelling outside Europe:

The time differences on long-haul journeys are for the most part so great that your wake-up time is outside the "time window" in which cyclotest® 2 plus accepts your measurement. In such cases please contact the cyclotest® service hotline in good time.

Turn off wake-up alarm

If you do not want to be disturbed, e.g. you want to have a lie-in at the weekend, you can turn off the wake-up alarm. To do so, press the large round Measurement button once briefly when the unit is switched off. You will see the following display:

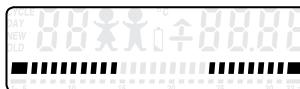
Fig. no. 26
Wake-up alarm on [more bars]

Again press the large round Measurement button once briefly.

Fig. no. 27
Wake-up alarm off [fewer bars]

To turn the wake-up alarm back on, proceed as described above. The unit confirms the input with a beep.

Measurement button pressed without sensor at point of measurement

cyclotest® 2 plus breaks off the measurement and signals this acoustically with three beeps and optically with an "E". See Fig. no. 13.

Start of cycle input incorrectly

If you have accidentally entered the start of your cycle incorrectly, call our service hotline. Keep your device to hand for this call.

Pregnancy display

cyclotest® 2 plus is waiting for your cycle start. In this case, there are two possibilities:

- 1) You have forgotten to start a new cycle on the first day of menstruation.
- 2) If there is no menstruation, you may be pregnant.

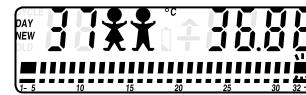
Fig. no. 28

Important: If within a cycle you observe inexplicable temperature responses which deviate from your previous cycles, call the cyclotest® service hotline.

cyclotest® 2 plus only indicates the fertile phase

- To be on the safe side, a longer fertile phase is indicated if:
- cyclotest® 2 plus still doesn't "know" enough about you, i.e. hasn't saved enough cycle values yet.
 - there are too few evaluable measurements available in one cycle.
 - your temperature is fluctuating too much due to e.g.: stress, illness, etc.

When to measure and how often?

Once the cycle start has been input, you can take your temperature measurements. Begin the morning after before getting up. What is important is that you have slept for at least 5 hours and you have not got up in the hour prior to measurement. No measurements are necessary during the infertile phase after ovulation. [No baby symbols are shown]

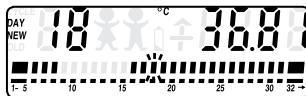
Fig. no. 25
You can now stop measuring until your next period.

If one or two measurements have been missed, this is not important. Simply continue measuring as normal on the next day. cyclotest® 2 plus continues counting the days automatically. If there are too few measurements that can be evaluated in a cycle, the unit, to be on the safe side, displays a longer fertile phase than usual.


Important: With each initial measurement in the cycle you determine the time window of +/- 2 hours for the daily measurement. You can therefore decide anew for each cycle whether you want to stick with the selected time or opt for a new time.

The female cycle

The female cycle is 28–29 days. Deviations of up to 5 days are considered to be within the normal standard range.

There are three distinct phases here:

- a relatively short infertile phase which begins with menstruation
- a fertile phase which ends shortly after ovulation
- a relatively long infertile phase after ovulation

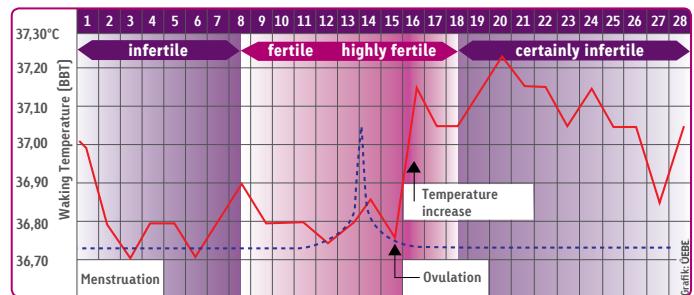The temperature method, which is based on the comparative measurement of the wake-up temperature during a woman's individual cycle phases, allows conclusions to be drawn about the processes in a woman's body. The more temperature values and information on the cycle duration are available, the narrower the start and end of the fertile phase can be pinpointed.

Failure and error messages

Failure encountered	Possible cause	Corrective action
Display "L"	<ul style="list-style-type: none"> - Sensor in the warm-up phase - Temperature too low - Possible technical fault 	<ul style="list-style-type: none"> - Continue measuring until end of measurement - Repeat measurement - If "L" is permanently displayed, contact cyclotest® service hotline
Display "H"	<ul style="list-style-type: none"> - Temperature too high - Fever - Illness - Possible technical fault 	<ul style="list-style-type: none"> - If "H" is permanently displayed without fever, contact cyclotest® service hotline
Display "E"	<ul style="list-style-type: none"> - Measured value not stable - Sensor removed before end of measurement 	<ul style="list-style-type: none"> - Repeat measurement - If "E" is repeatedly displayed, contact cyclotest® service hotline.

Customer service

The unit may only be repaired by the manufacturer or by an expressly authorised organisation.

Please contact:

UEBE Medical GmbH
 Zum Ottersberg 9
 97877 Wertheim, Allemagne
 Phone: +49 [0] 9342/924040
 Fax: +49 [0] 9342/924080
 E-mail: info@uebe.com
 Internet: www.uebe.com

If you are returning a unit from a non-EU country, please observe your national customs regulations.

Technical data

Unit type:	Cycle temperature computer for recording basal body temperature
Method:	Syntothermal method modified on the basis of the recognised regulations of the WHO
Unit dimensions:	8 cm x 10.8 cm x 3.3 cm
Weight:	approx. 110 g
Housing material:	PVC-free, ABS-base plastic
Temperature sensor:	PVC-free, TPE-base flexible sensor
Measurement cable:	Special cable with strain-relief device (kevlar)
Sensor:	Thermistor
Display:	LCD display (liquid crystal display)
Display elements:	Day, temperature value in °C, baby symbols, battery symbol, time-axis display, fault indications, direction arrow
Temperature display:	Absolute in °C (non-extrapolating)
Measurement range:	35.5 – 38.0 °C
Measurement accuracy under operating conditions:	±0.1 °C at ambient temperature 10 °C to 35 °C, rel. air humidity up to 93 % non-condensing, air pressure 700 to 1040 hPa
Storage and transport conditions:	Ambient temperature -10 to +50 °C, Rel. air humidity up to 93 % (non-condensing), Protect against moisture/humidity
Serial number:	The unit has a serial number which is identified with <u>SN</u> ; please quote this number in the event of possible service enquiries.

Battery:	RENATA type CR2450N mercury-free button-cell battery for approx. 500 measurements, battery to be replaced only by UEBE Medical GmbH. The unit may only be repaired by the manufacturer or by an expressly authorised organisation. Do not open the unit, as this will invalidate the warranty.
Service:	Should your unit show evidence of a fault, we ask you not to attempt repairs yourself, but instead to send the unit with a short description of the fault directly to UEBE Medical GmbH for inspection, as unit parts cannot be repaired individually.
Calibration:	The unit does not have to be calibrated; the metrological accuracy of the unit will be automatically checked when the battery is replaced by UEBE Medical GmbH.
Protection class IP:	IP 20: Protected against solid foreign particles with a diameter of more than 12.5 mm, no protection against water.
Expected service life:	5 years

Additional equipment connected to medical electrical equipment must comply with the respective IEC or ISO standards [e.g. IEC 60950 for data-processing equipment]. Furthermore, all configurations shall comply with the normative requirements for medical systems [see IEC 60601-1-1 or Section 16 of the 3rd edition of IEC 60601-1, respectively]. Anybody connecting additional equipment to medical electrical equipment is a system configurer and is therefore responsible for ensuring that the system complies with the normative requirements for systems. Attention is drawn to the fact that local laws take priority over the above-mentioned normative requirements. If in doubt, consult your local specialised dealer or the Technical Service department.

Explanation of symbols

Degree of protection against electric shock: TYP BF

Please observe operating instructions!

This product complies with the Council Directive 93/42/EC from 5 September 2007 regarding medical devices and bears the mark CE 0123 [TÜV SÜD Product Service GmbH].

Storage and transportation conditions
Ambient temperature -10 to +50 °C

Protect against moisture/humidity
Relative air humidity up to 93 %

Keep dry

Manufacturer

Disposal

Batteries and technical appliances must not be disposed of with domestic waste, but should be handed in at the appropriate collection and disposal points.

Cleaning and maintaining the unit

The flexible part of the sensor is waterproof, the rest of the sensor and the unit may only be wiped with a moistened cloth.

The device is not waterproof and must not be immersed in water! Clean the unit with a moistened cloth and a mild cleaning agent. You may use ethyl alcohol [available from all good pharmacies] to disinfect the sensor.

Do not expose the thermometer to extreme heat [blazing sun, heating, hot water, etc.].

Warranty

The device has been manufactured and tested with great care. However, in the unlikely event of a defect being detected after delivery, we provide warranty for new devices in accordance with the following terms and conditions:

1. During the warranty period of 2 years from the date of purchase we reserve the right either to repair any such defect at our expense or to supply a perfect replacement unit. The cost of returning the unit to our factory shall be borne by the sender. UEBE shall refuse to accept return deliveries that have not been paid for by the sender.
2. Excluded from the warranty are parts subject to normal wear and tear as well as damage caused by non-compliance with the instructions for use, improper handling [e.g. unsuitable power sources, breakages, leaking batteries] and/or disassembly of the unit by the purchaser. and/or disassembly of the unit by the purchaser. Furthermore, no claims for damages against us are substantiated by the warranty.
3. Warranty claims can only be advanced in the warranty period and by presenting proof of purchase. In the event of a warranty claim, the unit must be sent to the following address together with the proof of purchase and a description of the complaint: UEBE Medical GmbH Service-Center, Zum Ottersberg 9, 97877 Wertheim Germany
The cost of returning the unit to our factory shall be borne by the sender. UEBE shall refuse to accept return deliveries that have not been paid for by the sender.
4. The statutory claims and rights of the buyer against the seller [claims for defect, manufacturer's liability, for example] are not restricted by this warranty.

Please note: In the event of a warranty claim it is essential to attach the proof of purchase.

The cyclotest® service hotline

Dear Reader,

Ensure you read our notes on using the cyclotest® 2 plus carefully and familiarised yourself with its Features.

Our description corresponds to the normal cycle response. While dealing with your cyclotest® 2 plus you may notice initially some readings and information in the dialogue display that appear unfamiliar to you, or you may have questions about handling the unit.

In such cases you can call our cyclotest® service hotline, which is open from Mondays to Thursdays from 8.00 am to 5.00 pm and on Fridays until 4.30 pm.

Your team of cyclotest® advisers is looking forward to taking your call.

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© Copyright 2014 UEBE Medical GmbH